

PRX-T₃₃[®]

WiQO

HOW TO INCLUDE BODY SKIN TREATMENT WHEN TREATING CELLULITE



PROTOCOL FOR USE

ADVANTAGES

Cellulite has a mixed inflammatory and vascular etiopathogenesis. The most established pharmacological treatment includes the on-site injection of substances that promote draining and restoration of volumes. These injections can take place at a dermal level (mesotherapy) or subcutaneous level (anti-lipodystrophic agents).

The association with PRX-T₃₃ has the advantage of improving skin trophism, increasing its firmness, while using the WiQo Anti-Dryness Firming Body Cream at home completes stimulation and favours self-massage in the area, further contributing to the draining process.

TREATMENT MODALITY COMBINED WITH ANTI-LIPODYSTROPHIC AGENTS

- 1) Cleanse the skin with P SOLUTION or a delicate cleanser and dry thoroughly.
- 2) Apply PRX-T₃₃ twice, following its specific technique.
- 3) Administer the injection of anti-lipodystrophic agents.
- 4) Apply PRX-T₃₃ again.
- 5) Apply the WiQo Anti-Dryness Firming Body Cream, which is also to be used at home.

TREATMENT MODALITY COMBINED WITH MESOTHERAPY

- 1) Cleanse the skin with P SOLUTION or a delicate cleanser and dry thoroughly.
- 2) Apply PRX-T₃₃, following its specific technique.
- 3) Carry out the mesotherapy session as indicated by the manufacturer.
- 4) Do not apply any product immediately after the treatment.

TREATMENT SCHEDULE AND POST-TREATMENT

WITH ANTI-LIPODYSTROPHIC AGENTS

We recommend weekly sessions according to this schedule:

First session: PRX-T₃₃ + Poly(amino acid) anti-lipodystrophic gel

Second session: PRX-T₃₃ + Poly(amino acid) anti-lipodystrophic gel

Third session: PRX-T₃₃ by itself

Fourth session: PRX-T₃₃ + Poly(amino acid) anti-lipodystrophic gel

Fifth session: PRX-T₃₃ by itself

WITH MESOTHERAPY

We recommend 5 weekly combined sessions.



AFTER APPLICATIONS

Left leg submitted to treatment with anti-lipodystrophic gel only

Right leg submitted to combined PRX-T₃₃ and gel treatment

Direct line with researching dermatologists and biologists for scientific questions, or enquiries regarding outpatient treatment:
scientificdiv.gpq@wiqo.eu

wiqo[®]
a GPQ brand

GPQ srl

Operational Headquarters: Via Martinelli, 16 – 34015 Muggia (Trieste) - Italy
Phone: +39 040 360408 - Fax: +39 040 761393 - gpq@wiqo.eu
www.wiqo.com